



# SKED-PAK™ I JUMPABLE MEDICAL RUCK

NSN: 6530-01-472-4889



Inspired by the most rigorous demands of our elite military medical groups, this medical bag system is both durable and flexible, suitable for use by military, paramedics, field medic, smoke jumpers, search and rescue units, and any person with need for a really tough bag. The system is modular, with internal components which are removable and transferable. The external bags come in two styles. The Sked-Pak™ I is a heavy duty backpack which is designed to use with the standard Military Alice pack frame and is designed specifically for use in parachute jumps. It comes complete with an integral single point release mechanism and leg straps necessary for jumping safely.



The Sked-Pak™ I Jumpable Medical Ruck includes a sewn-on single point release and removable leg straps to meet the technical requirements for use by a parachutist. This design eliminates the need for the traditional "H" harness. When released the pack drops on a lowering line for personnel safety on landing. It includes outer storage pockets, two canteen pockets, and a removable accessory pack which can be attached on the top or bottom. A second accessory pack can be purchased to increase storage capacity.



## SKED-PAK™ I JUMPABLE MEDICAL RUCK INSTRUCTION



1. Attach an Alice Pack frame to the back of the Sked Pak I.



2. Find the lowering line attachment located on the top rear of Sked Pak I above the pack frame and attach the lowering line using a girth hitch then attach the other end of lowering line to parachute harness.



3. Secure Attachment Straps to the heavy webbing of the single point release.



4. Secure the loops and pass the 3rd loop through the grommet hole of the leg strap.





5. Insert the release cable through the 550 cord loop on both sides.



6. Secure the single point release Velcro to the Sked Pak I.



7. Pull leg straps around legs and fasten buckles.




8. Secure attachment straps to parachute harness and adjust them.



9. Tighten leg straps.



10. Sked Pak I is now upside down and properly attached to the jumper ready for exit from the aircraft.
11. Upon deployment of parachute canopy and with sufficient distance from other jumpers pull the grey single point release handle to release Sked Pak I from harness. Once released it will deploy the lowering line to full length and will impact the ground before the jumper.

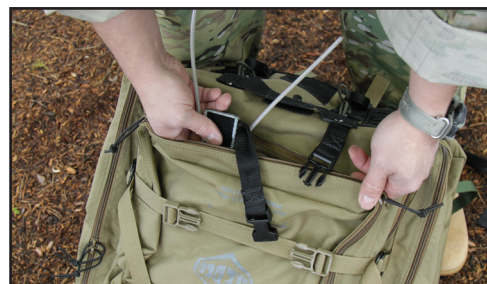

12. Once on the ground disconnect the single point release and stow it in a pocket of Sked Pak I.

**DECADES OF MILITARY SERVICE.**

MISSION CRITICAL SOLUTION FOR CASEVAC, MEDEVAC AND TACTICAL RESCUE SITUATIONS.

**800-770-7533**

**SKEDCO.COM**